Asus VivoWatch BP User Manual

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¢

•

Table of Contents

Bookmarks

•

Download this manual

Quick Links

A15521 June 2019 First Edition HC-A04 / HC-A04A

User Manual

Android App

iOS App

Table of Contents

Next Page

Related Manuals for Asus VivoWatch BP

Watch Asus VivoWatch BP User Manual (426 pages) Fitness Trackers Asus VivoWatch BP HC-A04 User Manual (52 pages) Watch ASUS VivoWatch User Manual (54 pages) Watch ASUS VivoWatch E-Manual (50 pages) Watch Asus VIVOWATCH E-Manual (48 pages) Watch Asus VivoWatch SP User Manual (324 pages) Watch Asus VivoWatch SP User Manual (188 pages) Watch Asus VivoWatch 5 User Manual (340 pages) Watch Asus VivoWatch 5 User Manual (196 pages) Watch Asus VivoWatch 5 AERO User Manual (54 pages) Watch Asus ZENWATCH3 E-Manual (64 pages) Watch Asus ZenWatch User Manual (58 pages) Watch Asus ZenWatch Quick Start Manual (19 pages) Watch Asus ZenWatch E-Manual (46 pages) Watch Asus ZenWatch 2 WI502Q E-Manual (62 pages)

Summary of Contents for Asus VivoWatch BP

Page 1 A15521 June 2019 First Edition HC-A04 / HC-A04A User Manual Android App iOS App...

<u>Page 3</u> Introduction Thank you for purchasing the ASUS VivoWatch BP! ASUS VivoWatch BP is the world's first wearable device that packs ASUS HealthAI technology and some fitness functions with all the benefits of being accurate, smart, and compact. Please fill in the following information for future reference:...

Page 4: About This Manual

Chapter 1: Getting started This chapter details the hardware components of your ASUS VivoWatch BP and shows you how to use the different parts of your ASUS VivoWatch BP. Chapter 2: Using your ASUS VivoWatch BP This chapter provides an overview of using your ASUS VivoWatch BP.

Page 5: Conventions Used In This Manual

This message contains important information that must be followed to keep you safe while performing tasks and prevent damage to your ASUS VivoWatch BP's data and components. Typography Bold = This indicates a menu or an item that must be selected.

Page 6: Safety Precautions

5°C and 40°C, and stored in environments with ambient temperatures between -20°C and 45°C. • DO NOT leave your ASUS VivoWatch BP in a hot car. Temperatures in a parked car on sunny days can exceed the recommended ambient temperatures.

<u>Page 7</u> • Keeping your ASUS VivoWatch BP and the strap clean and dry will reduce the possibility of skin irritation. • If you experience redness, swelling, itchiness, or any other irritation or discomfort on your skin around, or beneath, your ASUS VivoWatch BP, please remove your ASUS VivoWatch BP and consult your physician before resuming wear.

Page 8: Battery Precautions

Battery precautions • Do not dispose of in fire. • Never attempt to short-circuit the battery. • Never attempt to disassemble and reassemble the battery. • Discontinue usage if leakage is found.

Page 9 Getting started...

<u>Page 10</u> Getting to know your ASUS VivoWatch BP • Your ASUS VivoWatch BP can be used as a standalone device for measurements and time adjustments. • Download and install the companion app for a complete overview of your health data. For more details, refer to the...

Page 11 • ASUS VivoWatch BP complies with IP67 for dust and water- resistant. Suitable for splashes, rain, snow, or showering. Not suitable for swimming, diving, snorkeling, scuba diving, or high speed watersports. • DO NOT use your ASUS VivoWatch BP in liquids with chemicals or salt water (such as sea water, swimming pools, or hot springs) as it may damage the charging connectors.

Page 12 For more details, refer to the Using your ASUS VivoWatch BP section in this manual. Touch screen display panel Use the touch screen display panel to operate your ASUS VivoWatch BP using gestures. For more details, refer to the Using your ASUS...

Page 13: Home Button

Home button Press this button on your ASUS VivoWatch BP to return to the home screen. You can also press this button to unlock your ASUS VivoWatch BP or enable other functions. Press and hold this button for two (2) seconds to power on your ASUS VivoWatch BP, or press and hold this button for fifteen (15) seconds to power off your ASUS VivoWatch BP.

Page 14 Charging your ASUS VivoWatch BP 1. Align and place the charging connectors on your ASUS VivoWatch BP onto the charging connectors in the charging dock. Stand Portable charging dock charging dock The stand charging dock comes bundled with the HC-A04 and HC-A04A models, and the portable charging dock comes bundled with the HC-A04A model only.

<u>Page 15</u> 2. Gently push your ASUS VivoWatch BP so that it is firmly seated inside the charging dock. Stand charging dock Portable charging dock...

<u>Page 16</u> • Do not short circuit the charging connectors on the dock. • Some power banks can not detect low current devices, and will therefore not be able to charge the ASUS VivoWatch BP.

Page 17 Using your ASUS VivoWatch BP...

Page 18: Installing The Companion App

Installing the companion app Pair your ASUS VivoWatch BP with the companion app to take advantage of all its features, such as activity data, pill reminders, path tracking, message and call notifications, and sleep and health recommendations. 1. On your phone, log in or sign up for a Google account or Apple ID.

Page 19: Pairing For The First Time

Pairing for the first time • Turn on the Bluetooth of your phone before pairing. • Ensure that the ASUS VivoWatch BP is within three meters from your phone. 1. Press and hold the home button for two (2) seconds to power on your ASUS VivoWatch BP.

<u>Page 20</u> Pairing with another device or pairing again • Turn on the Bluetooth of your phone before pairing. • Ensure that the ASUS VivoWatch BP is within three meters from your phone. 1. From the Today screen, tap Device on the bottom of the screen.

Page 21 2. From the Device screen, tap Pairing. 3. Select your ASUS VivoWatch BP from the available list, then tap OK to start the pairing.

<u>Page 22</u> Navigating your ASUS VivoWatch BP Unlocking your ASUS VivoWatch BP Press the Home button to unlock your ASUS VivoWatch BP. You can scroll through the ASUS VivoWatch BP features after it is unlocked. After a period of inactivity, the ASUS VivoWatch BP...

<u>Page 23</u> Scrolling through ASUS VivoWatch BP features Swipe left or right to scroll through the ASUS VivoWatch BP features. You can press the Home button to return to the HOME screen.

<u>Page 24</u> Viewing more information for the selected feature Swipe up or down to view more information for the selected ASUS VivoWatch BP feature or change the watch face. Not all features contain additional information.

Page 25: Home Screen

The screens shown in this manual are for reference only. Home screen The ASUS VivoWatch BP home screen displays the time, date, and status of your ASUS VivoWatch BP. Icon...

Page 26: Measuring Screen

Measuring screen This screen allows you to measure your heart rate and health data. Your ASUS VivoWatch BP can also estimate your emotional stress by using the PTT (Pulse Transit Time) Index and the estimated the blood flow rate. Swipe right from the Home screen to enter the Measuring screen.

<u>Page 27</u> To get more accurate results: • Wear the ASUS VivoWatch BP in a normal way with the watch face facing up, the bottom of the ASUS VivoWatch BP touching your skin, and the ASUS VivoWatch BP away from your wrist bone.

Page 28 VivoWatch BP automatically determines your current heart rate. To get more accurate results: 1. Wear the ASUS VivoWatch BP in a normal way with the watch face facing up, the bottom of the ASUS VivoWatch BP touching your skin, and the ASUS VivoWatch BP away from your wrist bone.

<u>Page 29</u> Enter your personal information correctly for more accurate health data. Language Change device language. Brightness Adjust the brightness of your ASUS VivoWatch BP. This effect is more noticeable in darker environments. About View more information about your device, such as firmware...

Page 30: Activities Screen

Activities screen Your ASUS VivoWatch BP will automatically record your daily activities and display them on this screen. Swipe left from the Home screen to enter the Activities screen. You can view your daily activities such as steps you have taken, calories you have burned, your sleep time, and your comfort sleep time as they are recorded.

Page 31: Exercise Screen

Exercise screen This screen allows you to keep track of your exercise sessions. When you start exercise mode, your ASUS VivoWatch BP will record information about the current exercise session and show the information as they are recorded. Entering exercise mode To enter exercise mode, swipe left or right to the EXERCISE screen, then tap START to enter exercise mode.

Page 32 From the exercise screen, swipe left or right to view information such as the real time heart rate, exercise time, calories, steps, and distance for the current exercise session. Exiting exercise mode Press the home button and tap END to exit exercise mode.

<u>Page 33</u> Notifications When you receive a call or message on your paired phone, your ASUS VivoWatch BP will vibrate and show you the phone number of the caller or a preview of the message. Reminders Your ASUS VivoWatch BP will also show you walk and pill reminders to aid...

<u>Page 34</u> Updating your ASUS VivoWatch BP Updating through the ASUS HealthConnect App 1. Launch the ASUS HealthConnect App. 2. From the Today screen, tap Device on the bottom of the screen. Device icon...

Page 35 3. From the Device screen, tap on the device you would like to update. 4. Tap Update. A pop-up message will appear when the firmware update is completed.

Page 37 000 0000ASUS000000000 0000000000 000 00 00 00 000000	
ASUS HealthAl	
Image: https://account.asus.com/product_reg.aspx	

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https://www.asus.com/vivowatch/ASUS-vivowatch-BP-HC-A044/HelpDesk_Manual/...

 Page 39
 Contract and a contraction according accor

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Page 105 Appendix...

<u>Page 106</u> Federal Communications Commission Interference Statement This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

Page 107 RF exposure warning The equipment complies with FCC RF exposure limits set forth for an uncontrolled environment. The equipment must not be co-located or operating in conjunction with any other antenna or transmitter. Compliance Statement of Innovation, Science and Economic Development Canada (ISED) This device complies with Innovation, Science and Economic Development Canada licence exempt RSS standard(s).

Page 108 Radio Frequency (RF) Exposure Information The radiated output power of the Wireless Device is below the Industry Canada (IC) radio frequency exposure limits. The Wireless Device should be used in such a manner such that the potential for human contact during normal operation is minimized.

Page 109 Directive 2014/53/EU. Full text of EU declaration of conformity is available at https://www.asus.com/support/. The Simplified EU Declaration of Conformity of this device has been updated. This device now complies with Radio Equipment Directive 2014/53/EU, and no

longer complies with R&TTE...

<u>Page 111</u> Global Environmental Regulation Compliance and Declaration ASUS follows the green design concept to design and manufacture our products, and makes sure that each stage of the product life cycle of ASUS product is in line with global environmental regulations. In addition, ASUS disclose the relevant information based on regulation requirements.

Page 112: Proper Disposal

Proper disposal Do not throw your ASUS VivoWatch BP in municipal waste. This product has been designed to enable proper reuse of parts and recycling. The symbol of the crossed out wheeled bin indicates that the product (electrical, electronic equipment and mercury-containing button cell battery) should not be placed in municipal waste.

Page 113: Notices For Removable Batteries

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Page 114: Nordic Lithium Cautions (For Lithium-Ion Batteries)

Nordic Lithium Cautions (for lithium-ion batteries) CAUTION! Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type recommended by the manufacturer. Dispose of used batteries according to the manufacturer's instructions. (English) ATTENZIONE! Rischio di esplosione della batteria se sostituita in modo errato.

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Page 116: Copyright And Warranty Information

Product warranty or service will not be extended if: (1) the product is repaired, modified or altered, unless such repair, modification of alteration is authorized in writing by ASUS; or (2) the serial number of the is defaced or missing.

Page 117: Limitation Of Liability

ASUS. In each such instance, regardless of the basis on which you are entitled to claim damages from ASUS, ASUS is liable for no more than damages for bodily injury (including death) and damage to real property and tangible personal property;...

Page 120 15060-0LE2B000...

This manual is also suitable for: