

TOSHIBA

Toshiba HT-201 Instruction Manual

Digital clinical thermometer

1
2

--

•

Bookmarks

[Download this manual](#)

Quick Links



Enlarged version

Model number

Digital Clinical Thermometer for household use **[For Oral Use Only]**

Instruction Manual

Product name
Digital Clinical Thermometer
Model number
HT-201

Generic name
Electronic thermometer
Supply voltage
DC 3 V (CR2032 battery x 1 pc)

Battery life
Approx. 1 year (under the usage condition of room temperature 23°C, measurement for 7 minutes 1 time/day, data transfer 1 time/day)

Temperature
Four-digit with °C display in increments of 0.01°C

Illustrations in this instruction manual are schematic figures of this Product.

Measurement
± 0.05°C (33.00 to 38.00°C)
precision
± 0.1°C (33.00 to 34.99°C, 38.01 to 42.00°C)
(measured in a constant temperature water tank at a normal room temperature of 23°C)

Measurement
Predictive-type (compatible with predictive method temperature measurement and actual temperature measurement)

[Thermometer Main Unit]

Operation button
Measurement area

Battery cover
See the reverse side of

[Stored State]
this document for how to replace a battery.

Storage case

- Make sure that the temperature is taken at the base of tongue.
- Please refrain from using this Product after eating or drinking.
- Please refrain from using this Product after exercise.

Thank you for purchasing this Digital Clinical Thermometer. To use this Digital Clinical Thermometer correctly and safely, please read all of the instruction manual provided by TOSHIBA carefully and keep in mind the before using this Product.

After reading the manual, please keep it at hand so that others can refer it at any time.

* Specialized smartphone application is available. Visit the following site to confirm.
http://www.toshiba.co.jp/healthcare/bb/en/index_e.htm

* The information in the Manual (except for product specification and design) may be changed without prior notice.

Measurement range
33.00 to 42.00°C

Usage Environment
Ambient temperature: + 10 to + 40°C
temperature and
Relative humidity: 30 to 85% RH
humidity

(without condensation)
Storage Environment
Ambient temperature: - 10 to + 50°C
temperature and
Relative humidity: 30 to 90% RH
humidity

(without condensation)

Main unit's weight
: Approx. 20 g (including battery)

Outline Dimensions
: Approx. 46 (W) x 99.5 (L) x 11 (T) mm

Accessories

Temporary CR2032 lithium battery

(Installed in the product): 1pc.

Storage case : 1pc.

Instruction manual : 1pc

Display

Body Temperature Transfer Symbol

Shows the state ready to transfer body temperature data to your smartphone.

Previous Value Memory Symbol

Shows the body temperature of a measured last time.

Remaining Battery Symbol

Shows the remaining battery level in four increments.

Predictive Symbol

Shows a temperature which is measured by the predictive function.

This section lists important precautions that users of this Product (and anyone else) should observe in order to avoid injury to you and any third person and damage to property.

■ Explanation of Labels

Indicates "a hazardous situation which, if not observed, could result in death or serious injury".

Indicates "a potentially hazardous situation which, if not avoided, may result in minor or moderate injury or cause property damage".

Do not allow children to use, and keep this Product out of reach from small children.

If you allow this Product to use small children, this Product may cause their injury.

Prohibited

Do not use this Product for any purposes other than taking your body temperature.

(Using this Product on other living things such as your pets or other animals may cause their injury.)

Do not take your body temperature anywhere other than in-oral (armpit, ear, etc.).

(Using this Product in areas other than the mouth may cause your injury or result an incorrect measurement.)

This Product is exclusively designed for oral use.

Do not share this Product among multiple persons.

(Attempting to do so could cause infectious disease.)

Do not disassemble, repair or modify this Product.

(Attempting to do so may cause trouble or damage of this Product and result an incorrect measurement.)

Prohibited

Illustrations in this instruction manual are schematic figures of this Product.

ManualsLib.com

⚠ WARNING

⚠ CAUTION

This Product is not intended to diagnose
diagnose by yourself and consult with a



⚠ WARNING

⚠ CAUTION

This Product is not intended to diagnose
diagnose by yourself and consult with a



ManualsLib.com

Do not take your body temperature by using this Product that has been stored outside the range of the storage environment temperatures as set the above.

(Storage outside the range of the storage environment temperatures may cause trouble.)

Do not use this Product in a place where a strong static discharge or electromagnetic radiation occurs, or near a magnet.

(Such usage may cause trouble or an incorrect measurement result.)

1. The warranty period is one year from the date of your purchase of this Product. If trouble occurs under your normal use to follow the Manual, Toshiba will replace such defective Product with a new or an equivalent product. Any documents evidencing your purchase of this Product, including purchase date and location must be provided to Toshiba for replacement. Keep such documents in a safe place.

2. Toshiba will in no event be responsible for the deletion of any data or application stored on this Product.

3. In case of the trouble this Product occurs within warranty period as set forth in the above, please contact Toshiba customer service.

■ Explanation of Graphic Symbols

Indicates "prohibited actions" within or near the graphic symbol using an appropriate picture or text.

Indicates "actions that must be undertaken for safety purposes" by using an appropriate picture or text within or near the graphic symbol.

Instruction
Do not bite or bite off the measurement area of this Product. (Attempting to do so may cause an accident such as accidental swallowing.) In case of accidental swallowing, consult with a medical doctor immediately.

Prohibited
Do not heat the battery or place the battery in a fire. (Attempting to do so may cause injury due to battery rupture or similar.) Do not attempt to self-diagnose or self-treatment based on the measurement result.

(Attempting to do so may cause worsening of symptoms.) Please consult with a doctor when seeking medical attention. Keep batteries or the battery cover out of reach from small children.

(To prevent small children from swallowing batteries or the battery cover accidentally.) If this Product is swallowed, consult with a medical doctor immediately.

Use a battery designated in the Manual. (Using an undesignated battery may cause injury and disaster due to heat generation, rupture, leakage, or ignition of the battery.) Set the battery in this Product correctly so that "+" and "-" are in their proper positions.

Instruction
(Incorrect setting of the battery may cause injury and disaster due to heat generation, rupture, leakage, or ignition of the battery.) Do not drop, step on or otherwise apply a strong shock to this Product or the storage case.

(Such usage may cause trouble or damage to this Product and result in an incorrect measurement.)

Be careful so that fluid or water does not enter into this Product. (This Product is not designed as fully waterproof. Entry of water into this Product may cause trouble of this Product.)

4. Even within warranty period as set forth in the above, in case that one of the following occurs, Toshiba provides you with repair or replacement service, subject to your payment of Toshiba's designated fee.


Failure or damage caused by improper use, storage, or this Product tampering and repair.

Failure or damage caused by drops or shock.

Failure or damage caused by disasters (fire, flood, earthquake) or a Force Majeure.




Failure or damage unrelated to this Product.

Consumables. (battery, storage case)

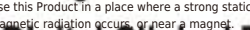

Digital Clinical Thermometer for household use [For Oral Use Only]

Instruction Manual

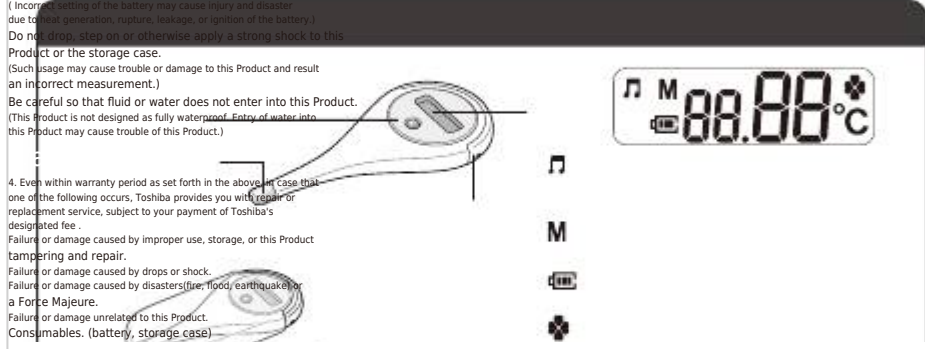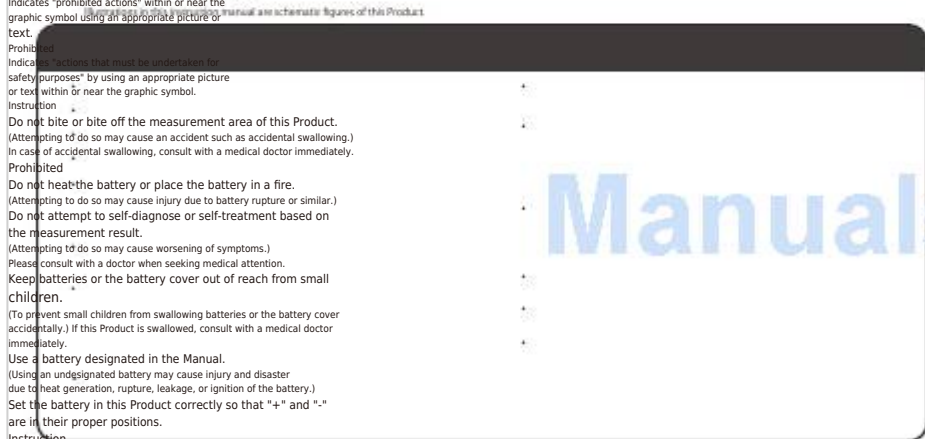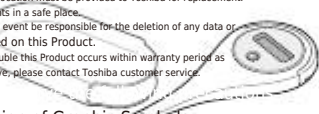

WARNING

CAUTION

This Product is not intended to diagnose or diagnose by yourself and consult with a



ManualsLib.com

[Next Page](#)

1

2

Summary of Contents for Toshiba HT-201

[Page 1](#) Failure or damage caused by drops or shock. 2. Toshiba will in no event be responsible for the deletion of any data or Failure or damage caused by disasters (fire, flood, earthquake) or

- Make sure that the temperature is taken at the base of tongue.

[Page 2](#) ● If “ ” is displayed, data transfer to your smartphone is available. temperature again after at least 30 seconds lapse. For details, following web site :
http://www.toshiba.co.jp/healthcare/bbt/en/index_e.htm Delete your body temperature data in this Product “Er.11 to 19” is displayed.